# CardioSecur Active

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User Manual (EN)

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Version V7.1

18.10.2018

© by Personal MedSystems GmbH

# Index

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E-Mail: [info@cardiosecur.com](mailto:info@cardiosecur.com)

## Index

|    |   |    |
|----|---|----|
| 1  | Additional Information                  | 28 |
| 2  | Symbols                                 | 28 |
| 3  | Intended Use                            | 31 |
| 4  | Safety Information                      | 34 |
| 5  | Connecting the CardioSecur ECG Cable    | 39 |
| 6  | Disposable Electrodes                   | 40 |
| 7  | Serial Number and Contact Address       | 43 |
| 8  | Technical Data and Operation Conditions | 45 |
| 9  | Care and Maintenance                    | 47 |
| 10 | Electromagnetic Compliance (EMC)        | 48 |
| 11 | Disposal                                | 51 |
| 12 | Labelling                               | 51 |
| 13 | Contact                                 | 52 |

## 1 Additional Information

This following user handbook describes the usage of CardioSecur Active on iPhone 5/5s/5C, 6/6Plus, 7/7Plus, 8/8Plus and X (iPhone is a registered trademark of Apple Inc.) as well as on chosen Android smartphones. The latest list is available for you at [www.cardiosecur.com](http://www.cardiosecur.com). For reasons of simplicity we refer herein exclusively to smartphone and CardioSecur.

Additional information can be found at [www.cardiosecur.com/en/active-manual](http://www.cardiosecur.com/en/active-manual).

## 2 Symbols

Packaging included in the scope of delivery or this user manual may display the following symbols (color and size can vary):

| Symbol  | Meaning   |
|---|---|
|  | Use disposable electrodes before the displayed date of expiry |
| <b>LOT</b>  | Lot denomination  |

| Symbol  | Meaning   |
|---|---|
|  | This product complies with the requirements of the directive for medical devices 93/42/EWG. |
| <b>REF</b>  | Item number   |
|  | Please refer to user manual   |
|  | Warning<br>Warning concerns safety-relevant information                                     |
|  | Attention<br>Attention concerns information of high relevance                               |
|  | Information<br>Information concerns details that are worth knowing                          |
|  | Manufacturer's data   |
|  | Store at indicated temperature with unbroken seal   |

| Symbol  | Meaning   |
|---|---|
|  | For single use only   |
|  | Latex-free  |
|  | CardioSecur and its accessories - like any other electronic device – contain metal and synthetic parts, which need to be disposed of separately after operating life in order to prevent negative environmental impact. Send CardioSecur and its accessories with sufficient postage, and if possible in its original packaging, to Personal MedSystems GmbH for disposal free of charge. |
|  | Application unit of type BF. An application unit of type BF grants protection against electric shock, in particular with regard to permissible discharge current, according to the requirements of norm EN 60601-1.   |

### 3 Intended Use


CardioSecur is intended to record, evaluate and store resting electrocardiograms (ECG) for adults.

CardioSecur is a medical electronic system, comprising an ECG cable and an app for a smartphone. CardioSecur can record, evaluate and store a complete 15-lead ECG using 4 electrodes. CardioSecur is able to detect arrhythmias and ischemia of the heart by comparing a reference reading to a control reading. CardioSecur serves the purpose of self-monitoring heart activity.

The user is required to record a personal reference reading when symptom-free. The reference reading is stored in the CardioSecur app. The user may perform the reading himself or under medical supervision. A control reading may be performed immediately in the case of symptoms or at regular intervals (e.g. every other day), independent of symptoms.

The user is not required to have previous medical knowledge or special training to use CardioSecur. CardioSecur is recommended for individuals with a known or suspected heart condition or risk factors for cardiac disease (e.g. hypertension, diabetes mellitus or increased cholesterol levels). Patients and individuals taking precautionary measures may use CardioSecur.

In order to record an ECG (reference and control ECG), disposable electrodes need to be positioned on the user's thorax according to the following diagram:

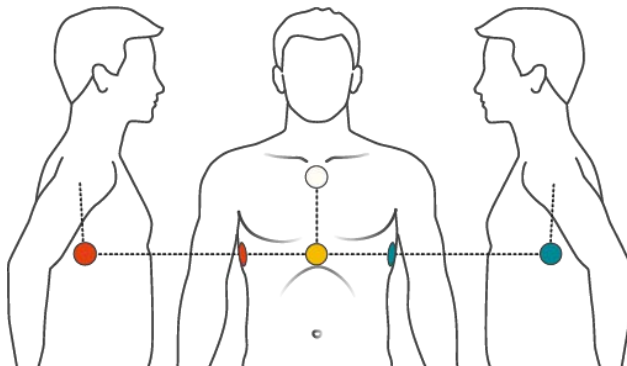White electrode: At the upper end of the sternum (on the bone)

Yellow electrode: At the lower end of the sternum (on the bone)

Red electrode: In the middle of the right axillae at the height of the lower end of the sternum (the same height as the yellow electrode)

Green electrode: In the middle of the left axillae at the height of the lower end of the sternum (the same height as the yellow electrode)

Prior to a reading the user is required to sit calmly and relax for a minimum of 3 minutes. During a reading (duration ca. 1 minute) the user must continue sitting calmly and relax without talking or coughing.

CardioSecur performs a comparative evaluation between the reference reading and a control reading and thereupon provides the user with a recommendation to act:

- **White result:** Indicates “no changes” compared to the personal reference ECG. There is generally no need for action.
- **Yellow result:** Indicates “minor irregularities” compared to the personal reference ECG. CardioSecur recommends planning a doctor’s visit for clarification.
- **Red result:** Indicates “severe irregularities” compared to the reference ECG. CardioSecur recommends seeing a doctor instantly.

In accordance with applicable guidelines of the European Society of Cardiology, CardioSecur recommends visiting a doctor instantly whenever symptoms or discomfort prevail for more than 20 minutes. This recommendation applies even when the result given by CardioSecur shows “no changes” or “minor irregularities” compared to the reference ECG.

The user may store, print and manage all ECG readings (reference and control readings) taken with CardioSecur by means of a personal user account on the CardioSecur database, an optional service by Personal MedSystems. In addition users may self-select a physician to view personal ECG data for informational purposes.

## 4 Safety Information

Take note of the following warnings and information. In case of non-compliance you run the risk of causing malfunction, damage or bodily injury.

**Warning:** Only use CardioSecur in accordance with its intended use (see section 3, Intended Use).

**Warning:** CardioSecur contains personalized data. It must only be used by the individual whose personal reference ECG is stored.

**Warning:** The results provided by CardioSecur are not to be used as the basis for beginning or changing therapeutic measures without an independent medical examination.

**Warning:** ECG readings by CardioSecur are not able to detect all changes indicative of cardiac disease! If you notice changes to your state of health, contact your physician instantly. In case of prevailing symptoms for more than 20 minutes, contact your doctor instantly, even if the results provided by CardioSecur show no or minor irregularities compared to your reference ECG.

**Warning:** Do not attempt to make a diagnosis on the basis of the results and analysis provided by CardioSecur. Always check with your physician. Self-diagnosis may cause a deterioration of your state of health or medical condition.

**Warning:** Results provided by CardioSecur reflect the moment of a recording. Your state of health may change rapidly. In case you notice a change in your state of health, contact a physician.

**Warning:** CardioSecur serves the purpose of detecting new ischemia and arrhythmias of the heart. CardioSecur is not designed for continuous ECG monitoring, but merely evaluates a short time window of your heart's activity.

**Warning:** Note that a physician may only receive your ECG recordings and your attached comments if you are actively linked to the physician through the CardioSecur database. Transmission of ECG data to the linked physician is not guaranteed. In addition, when your ECG data is received by your physician, this does not guarantee that he or she will look at the data or establish a diagnosis. It is recommended that you contact the physician you are linked to directly. This is especially important in the case of an emergency.

**Warning:** Before performing a reading with CardioSecur sit calmly for a minimum of 3 minutes. Ensure that your posture is relaxed. Being seated with a straight back is ideal. Do not cough or speak during a reading. Body movements may cause interference influencing the evaluation.

**Warning:** Do not perform readings when subjected to vibrations, e.g. in a moving vehicle, train or plane.

**Warning:** Do not perform ECG readings if the attachment points of the electrodes on your skin are wet, e.g. due to sweat or after bathing.

**Warning:** Proper functioning of the disposable electrodes is influenced by large amounts of body hair, cream or oily skin. Regularly remove hair from your skin at the attachment points of the electrodes and ensure that your skin is free of cream and fat. Do not clean your skin with alcohol.

**Warning:** Do not record a reference ECG with incorrectly positioned or transposed electrodes, as a correct evaluation of subsequent control ECGs is not possible.

**Warning:** Renew your reference ECG after every acute event, intervention (e.g. heart attack, cardioversion in case of arrhythmia, heart bypass intervention, heart pacemaker implantation, cardiac catheter intervention, heart valve intervention), medication change or when recommended by your physician.

**Warning:** Only use the disposable electrodes recommended by Personal MedSystems (YD43-6-W or P-00-S/xx).

**Warning:** Only use disposable electrodes that are unused, have not expired and are at room temperature, in order to ensure sufficient signal quality and prevent distorted results.

**Warning:** Remove all disposable electrodes after performing a reading, as prolonged adhesion may cause skin irritation.

**Warning:** Do not use CardioSecur in case of sensitive skin or allergies. This could cause rash, irritation of the skin or eczema.

**Warning:** Only use the ECG cable as described in this user manual and provided by Personal MedSystems GmbH.

**Warning:** Do not use CardioSecur when using an external defibrillator (CardioSecur is not defibrillator proof).

**Warning:** During use of CardioSecur, do not connect any other device or computer via an adapter to your smartphone and/or to the headset jack of your smartphone.

**Warning:** Do not perform readings in places exposing CardioSecur to high electromagnetic radiation (e.g. TV sets, monitors, dishwashers, dimmer).

**Warning:** Do not expose CardioSecur to any electrostatics. Discharge any electrostatics out of your body before touching CardioSecur.

**Warning:** Do not expose CardioSecur to any physical shocks or vibrations. Do not drop it on the floor or step on it.

**Warning:** Do not place anything onto CardioSecur, as this may damage the device.

**Warning:** Do not open, take apart or repair CardioSecur. Do not transform CardioSecur. CardioSecur does not contain any parts that need to be maintained by the user.

**Warning:** Store and use CardioSecur only under the conditions described under the section 8, Technical Data and Operation Conditions.

**Warning:** Do not sterilize CardioSecur in a steam sterilizer (autoclave) or in a gas sterilizer (ethylene oxide, formaldehyde, ozone etc.).

**Warning:** Do not use CardioSecur in the vicinity of combustible anaesthetics, drugs or pressurized oxygen.

**Warning:** Only use CardioSecur if it has no apparent damage.

**Warning:** Note the information in section 9, Care and Maintenance.

**Warning:** CardioSecur utilizes an ECG method computing a 15-lead ECG from 3 leads read with 4 electrodes. Deviations might arise in comparison to a 12-lead ECG read with 10 electrodes.

**Attention:** Ensure that date and time in your smartphone are set correctly.

**Attention:** Contact our customer support if you are unable to perform a firmware update.

## 5 Connecting the CardioSecur ECG Cable

Connect the CardioSecur ECG cable to your smartphone.

|   |  |
|---|--|
|    |    |
| <p>A CardioSecur with 30-Pin connector can be plugged directly into an iPhone with 30-Pin jack (e.g. iPhone 4S).</p>      | <p>A CardioSecur with 30-Pin connector can be plugged into an iPhone with Lightning jack (e.g. iPhone 7) only by means of the Apple Lightning adapter.</p> |
|    |    |
| <p>A CardioSecur with Lightning connector can be plugged directly into an iPhone with Lightning jack (e.g. iPhone 7).</p> | <p>A CardioSecur with micro-USB connector can be plugged directly into an Android smartphone (e.g. Samsung Galaxy S7).</p>                                 |

## 6 Disposable Electrodes

Disposable electrodes are the link between your body and CardioSecur. They are of critical importance for the technology's proper function. Personal MedSystems GmbH cooperates with highly specialized manufacturers for disposable electrodes in order to ensure optimal signal quality.

### Intended Use

YD43-6-W and P-00-S/xx are disposable electrodes designed for recording electrocardiograms (ECGs).

**Warning:** Only use disposable electrodes if the expiry date has not passed.


**Warning:** Only use disposable electrodes if they have reached room temperature.

**Warning:** Only store disposable electrodes with an unbroken seal and at temperatures between 10 °C and 25 °C.

### Operating Instructions


Use disposable electrodes YD43-6-W or P-00-S/xx according to the subsequent steps:

1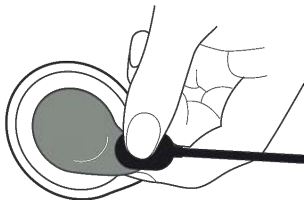

First connect the 4 ECG electrode connectors to an unused disposable electrode.

2

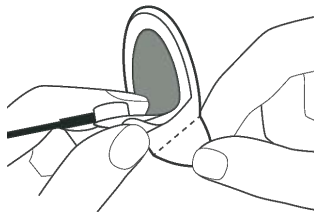Remove the protective foil from the disposable electrodes and attach them without touching the adhesive side and without putting pressure on the middle part (see diagram 3, Intended Use).

3

Remove the disposable electrodes carefully from your skin after performing a reading. Careless removal may cause damage to your skin. Should you encounter difficulties in removing the disposable electrodes, dissolve the adhesive agent of the electrodes with water. Remaining gel may be removed from your skin with paper tissues or water.

Note the instructions provided on the backside of the disposable electrodes' pouch.

### Manufacturer Information

Ambu A/S  
Baltorpbakken 13  
DK-2750 Ballerup  
Denmark

Made in Malaysia

P-00-S/xx

Yancheng Tianrun Medical Instrument Factory  
Hequiao Industrial Area, West Qingnian Road  
Yancheng City, Jiangsu Province  
China

Made in China

YD43-6-W


## 7 Serial Number and Contact Address

1

Press the „Menu“ icon (☰).

2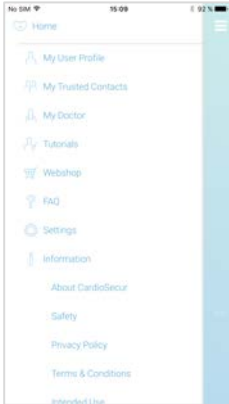

Press „Information“ and then "About CardioSecur".

3

The following information is displayed on the information page:

- Software version of your iPhone/Android app
- Hard- and software version of your CardioSecur ECG cable
- Serial number of your CardioSecur ECG cable
- Personal MedSystems GmbH contact details
- Product certification

## 8 Technical Data and Operation Conditions

| Parameter                          | Description                                      |
|------------------------------------|--|
| Model                              | CardioSecur Active                               |
| Lead method                        | Bipolar, 3 lead                                  |
| Reading quality                    | 15-lead ECG with 4 electrodes                    |
| Pulse range                        | 18 to 256 beats/minute                           |
| Precision                          | Band width: 0,05 to 40 Hz, sampling rate: 250 Hz |
| Type of protection                 | IP22   |
| Storage                            | 10,000 readings                                  |
| Electrical safety                  | Application unit type BF                         |
| Power supply                       | through Smartphone                               |
| Temperature and humidity operation | +5 °C to +40 °C, 10 % to 93 %, non-condensing    |
| Temperature and humidity storage   | -25 °C to +70 °C, 10 % to 93 %, non-condensing   |
| Air pressure for operation         | 700 to 1060 hPa                                  |
| Electrodes                         | Disposable electrodes, Typ YD43-6-W or P-00-S/xx |

**Warning:** Do not store CardioSecur under the following conditions:

- Direct sunlight
- Wet or damp places
- Dusty places
- In the vicinity of fire or open flames
- Places subject to physical shocks
- Places subject to high electromagnetic radiation

**Information:**

- Keep the original packaging of CardioSecur and its accessories in order to identify the product in the case of recall.
- We reserve the right to change technical data without prior notice.
- No safety-related controls are necessary during the product's lifetime.

## 9 Care and Maintenance

### Cleaning and Maintenance of the smartphone

Please follow the instructions from the manufacturer to clean and maintain your smartphone.

### Cleaning and Disinfection of CardioSecur

Clean your CardioSecur with a slightly dampened towel. In most cases cleaning with water and soap will be sufficient. Liquids on the casing must instantly be wiped off with a dry towel.

**Warning:** Never clean CardioSecur with benzene, benzine, paint thinner or concentrated alcohol.

To disinfect CardioSecur, wipe it off with a disinfectant. For disinfecting CardioSecur use a non-aggressive solution. Recommended disinfectants (for superficial disinfection as recommended in the respective manufacturer's user manual) are Helipur® H plus N and Sagrotan® med concentrate.

**Warning:** Never dip CardioSecur or its parts into a cleaning liquid, and do not sterilize CardioSecur in a steam sterilizer (autoclave) or in a gas sterilizer (ethylene oxide, formaldehyde, ozone etc.).

### Maintenance of CardioSecur

**Warning:** Do not open CardioSecur. CardioSecur does not contain any parts that need to be maintained by the user.

## 10 Electromagnetic Compliance (EMC)

**Warning:** Note important information on electromagnetic compliance (EMC). The growing number of electronic devices such as PCs, displays, TV sets and washing machines may subject medical devices to electromagnetic interference. This may cause malfunctioning of the medical device and create potentially unsafe situations. Medical devices should also not interfere with other electronic devices. The norm IEC60601-1-2 was introduced to regulate the requirements for EMC and prevent unsafe product situations. This norm defines the degrees of immunity against electromagnetic interference and the maximum electromagnetic emission levels for medical devices. CardioSecur complies with norm IEC60601-1-2:2001 concerning immunity as well as emissions. Nonetheless, special precautions should be taken. Do not use devices that cause strong electric or electromagnetic fields in the vicinity of CardioSecur. **A minimum distance of 2 meters is recommended. If the distance is less than 2 meters CardioSecur may not perform properly.** In accordance with the norm IEC60601-1-2:2001, contact our customer support (see section 13, Contact) to request further documentation.

**DIN EN 550112**

Industrial, scientific and medical high frequency devices (ISM devices)

Radio frequency interference – limit values and measuring procedure

| Interference emission                       | Limit value class | Result |
|---|-------------------|--------|
| Interference voltage                        | --                | n/a    |
| RFI field strength (not accredited testing) | B                 | OK     |

**DIN EN 60601 1-1-2**

Medical electrical equipment

Part 1-2: General requirements for safety:

Collateral standard: electromagnetic compliance; requirements and testing

**DIN EN 60601 1-2-47**

Medical electrical equipment

Part 2-47: Specific requirements for safety including essential performance characteristics for outpatient electrocardiographic systems

| Interference immunity against  |                                      | Testing accuracy    |                     | Result |
|--|--------------------------------------|---------------------|---------------------|--------|
|  |                                      | normative           | tested              |        |
| Static electrical discharge  | air                                  | 8 kV                | 8 kV                | OK     |
|  | contact                              | 6 (8) kV            | 8 kV                | OK     |
| High frequency electromagnetic fields<br>(80 – 2500 mHz)   | housing                              | 3 V/m               | 10 V/m              | OK     |
| Fast transient disturbances (chest)  | mains                                | 2 kV                | 2 kV                | n/a    |
|  | I/O                                  | 1 kV                | -                   | n/a    |
| Surge voltages   | unsym.                               | 2 kV                | -                   | n/a    |
|  | sym.                                 | 1 kV                | 1 kV                | n/a    |
| Conducted disturbances, induced by high frequency  | mains                                | 3 V                 | 10 V                | n/a    |
|  | I/O                                  | 3 V                 | -                   | n/a    |
| Energy-related frequency – Magnetic field 50 Hz<br>(Helmholtz coil)  | housing                              | 3 A/m               | 3 A/m               | n/a    |
| Voltage dips, short-term interruptions<br>Test levels as % of UT duration in supply<br>frequency cycles<br>All testing at 100 V and 240 V each 50 Hz | mains UT<br>100 – 240V<br>50 / 60 Hz | <5 %<br>0,5 cycles  | <5 %<br>0,5 cycles  | n/a    |
|  |                                      | 40 %<br>100 cycles  | 40 %<br>100 cycles  | n/a    |
|  |                                      | 70 %<br>500 cycles  | 70 %<br>500 cycles  | n/a    |
|  |                                      | <5 %<br>5000 cycles | <5 %<br>5000 cycles | n/a    |

## 11 Disposal

**Warning:** Ensure proper disposal of CardioSecur (electronic waste).

Labelling on CardioSecur, its packaging, and this user manual indicate that CardioSecur must not be disposed together with ordinary household refuse after its lifetime. Dispose of CardioSecur separately, so that harm to the environment or human health is not caused by uncontrolled disposal.

## 12 Labelling

[www.cardiosecur.com](http://www.cardiosecur.com)  
Personal MedSystems GmbH  
Wilhelm-Leuschner-Str. 41  
60329 Frankfurt am Main  
Made in Germany 201x

## 13 Contact

Personal MedSystems GmbH

Wilhelm-Leuschner-Str. 41

60329 Frankfurt am Main

Germany

Phone: +49-(0)69-9072013-0

Fax: +49-(0)69-9072013-99

E-mail: [info@cardiosecur.com](mailto:info@cardiosecur.com)

Website: [www.mobile-ecg.com](http://www.mobile-ecg.com) / [www.cardiosecur.com](http://www.cardiosecur.com)

Refer to our customer support in case of queries regarding CardioSecur.

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